IT

Polpaccera Vulkan Classic 3007

Ideale per

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These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

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Manufactured for
Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
USA
www.performancehealth.com

Performance Health ANZ
Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France
13 rue André Pingat
CS 10045
51724 Reims cedex
France
Tel: 00(33) 03 10 00 79 30
Fax: 00(33) 03 10 00 79 49

**PERFORMANCE
HEALTH**

VULKAN

Vulkan Classic 3007 Calf Support

CE

Ideal for

- Suitable for calf strains and shin splints
- A tough compression sleeve ideal for reducing pain associated with calf strains and shin splints

Hazards & Cautions

Do not wear if warming or heat-generating liniments, ointments or creams have been applied.

Should an irritation or rash develop, discontinue use immediately and seek medical advice.

If your injury is severe or does not improve within three to four days seek medical advice.

Do not use in case of signs of wear and/or damage to the support

Ensure that the support is suitable for the user

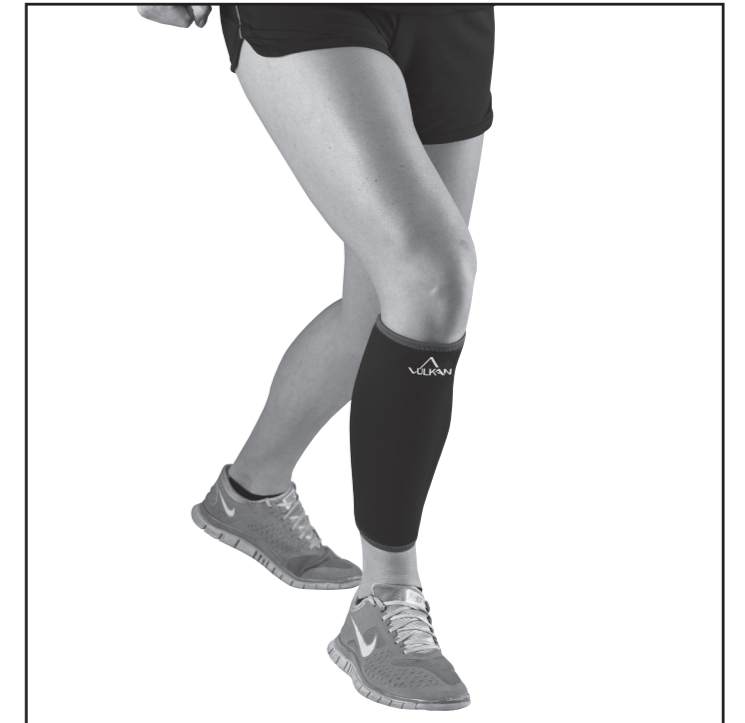
Cleaning & Care

- Hand wash only below 40°C
- Do Not dry clean
- Do Not bleach
- Do Not iron
- Do Not tumble dry

Fitting Instructions

The support should be sized correctly to ensure maximum benefit. To determine the correct size measure the circumference of the calf at the widest point.

Ensure that the support is not uncomfortably tight and does not restrict blood flow.

Available sizes

Code	Size	Dimensions
09 156 7940	S	30 - 34cm / 12 - 13½"
09 156 7957	M	34 - 38cm / 13½ - 15"
09 156 7965	L	38 - 42cm / 15 - 16½"
09 156 7973	XL	42 - 46cm / 16½ - 18"

Material Composition

80% Neoprene, 15% Nylon, 5% Elastic.
Contains natural rubber.

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