IT

## Gomitiera Vulkan Classic 3074

### Ideale per

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These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

**Manufactured for**  
**Performance Health International Ltd.**  
Nunn Brook Road, Huthwaite,  
Sutton-in-Ashfield, Nottinghamshire,  
NG17 2HU, UK.

**UK:**  
Tel: 03448 730 035  
Fax: 03448 730 100  
www.performancehealth.co.uk

**International:**  
Tel: +44 1623 448 706  
Fax: +44 1623 448 784

**Performance Health Supply, Inc.**  
W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012  
USA  
www.performancehealth.com

**Performance Health ANZ**  
Unit 3, 3 Basalt Road,  
Pemulwuy NSW, Australia 2145.  
Tel: 1300 473 422  
Fax: 1300 766 473

**Performance Health France**  
13 rue André Pingat  
CS 10045  
51724 Reims cedex  
France  
Tel: 00(33) 03 10 00 79 30  
Fax: 00(33) 03 10 00 79 49

**PERFORMANCE  
HEALTH**

## Vulkan Classic 3074 Tennis Elbow Strap

### Ideal for

- Suitable for use during activities where repetitive strain injury may be an issue such as golf, racquet sports or manual occupations
- Removable strap provides additional compression if required

### Hazards & Cautions

Do not wear if warming or heat-generating liniments, ointments or creams have been applied.

Should an irritation or rash develop, discontinue use immediately and seek medical advice.

If your injury is severe or does not improve within three to four days seek medical advice.

Do not use in case of signs of wear and/or damage to the support

Ensure that the support is suitable for the user

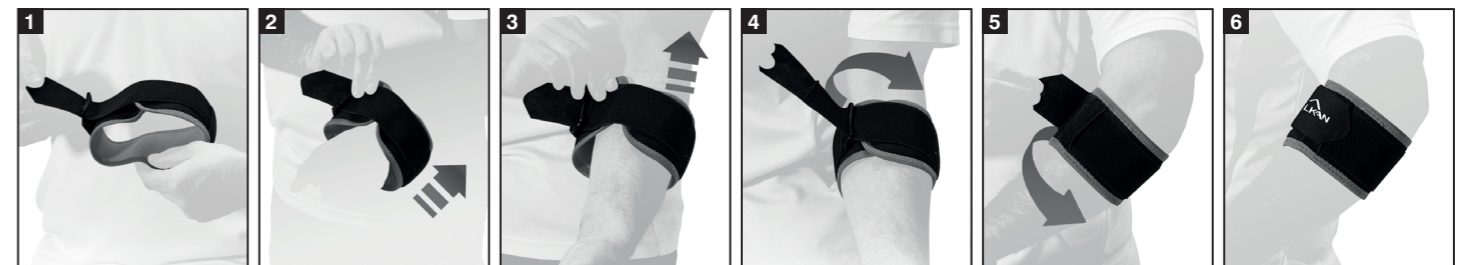
### Cleaning & Care

- Hand wash only below 40°C
- **Do Not** dry clean
- **Do Not** bleach
- **Do Not** iron
- **Do Not** tumble dry

### Fitting Instructions

The support should be sized correctly to ensure maximum benefit. To select the correct size measure the circumference around the elbow joint with the arm held straight.

Ensure that the support is not uncomfortably tight and does not restrict blood flow.

### Available sizes

09 156 7254

Fits all sizes

### Material Composition

Neoprene 80%, Nylon 7%, Elastic 6%, Polypropylene 4%, Polyester 3%.  
Contains natural rubber.

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